



IsoBind Blood and Tissue kit

Cat No. IB-BTDNA-100

System: Silica spin columns (manual workflow)

Sample types: Whole blood, buffy coat, body fluids, fresh tissue, frozen tissue

USER MANUAL

Website www.gene-vantage.com

Technical support info@gene-vantage.com

In vitro diagnostics

Table of Contents

1. Kit Contents	3
2. Important Notes	5
3. Safety Precautions	7
4. Kit Principles	8
5. Hardware And Consumables (Supplied By The User)	9
6. Quick View Protocol	10
7. Kit Specifications	12
8. Workflow Tips	12
9. Preparing Buffers And Equipment	13
10. Complete Protocol	15
11. Troubleshooting Guide	16
12. Product Use Restriction / Warranty	18

Technical support: info@gene-vantage.com

1. KIT CONTENTS

The Blood and Tissue DNA Extraction Kit from Gene Vantage is meticulously designed for the isolation of high-quality DNA from various blood and tissue samples. This kit incorporates state-of-the-art components and reagents, optimised for efficiency, ease of use, and compatibility with both manual and automated systems. Each component is tailored to ensure optimal performance across a broad range of biological samples, making it a superior choice for researchers and diagnostic professionals.

Component	Description/ Function	Volume Required per Sample	Short Term Storage	Long Term Storage	Total for 100 Samples	Total for 250 Samples
Lysis Buffer	Lyses cells to release DNA. Specially formulated to ensure complete lysis of cellular material for optimal yield.	400 μ L	Room temperature	Room temperature	40mL	100mL
Binding Buffer *Add Isopropanl (95%) prior to use	Facilitates binding of DNA to the silica membrane.	500 μ L	Room temperature	4-8°C	50mL	125mL
Wash Buffer A * Add EtOH (96-100 %) prior to use	Removes impurities such as cellular debris and proteins without stripping away the bound DNA	700 μ L	Room temperature	Room temperature	70mL	175mL

Wash Buffer B * Add EtOH (96-100 %) prior to use	Removes residual salts without stripping away the bound DNA	700 µL	Room temperature	Room temperature	70mL	175mL
Elution Buffer	Elutes purified DNA from column	50 µL	Room temperature	4-8Â°C	5mL	12.5mL
Protease K	Enhances lysis by breaking down proteins, facilitating more efficient DNA release.	20 uL	Room temperature	-20Â°C	2 mL	5 mL
Silica Spin Columns	Silica matrix that selectively binds DNA while allowing other compounds to pass through it	1 column + collection tube	Room temperature	Sealed in ziplock at 4-8Â°C	100 units	250 units



Buffers contain skin irritants



Wear gloves

2. IMPORTANT NOTES

Before beginning your work with the Gene Vantage IsoBind Blood and Tissue Kit, please take a moment to review these important notes. Adhering to these guidelines will ensure optimal results and efficiency throughout your extraction process.

Sample Preparation: Achieving a homogeneous sample is crucial for consistent DNA yields.

Tissues should be finely minced using clean, sharp scissors or homogenized in a bead mill or tissue disruptor specific to the sample type (e.g., muscle, liver). For blood samples, ensure thorough mixing to prevent clotting before processing.

Handling of Samples: Biological samples should be handled with care to prevent degradation of DNA. Keep samples on ice when possible during preparation and process them promptly after collection to minimize DNA breakdown. For samples high in protein or fat (such as liver tissue), increase the lysis time and ensure thorough mixing. This aids in complete disruption of cellular structures, releasing more DNA into the solution.

Buffer Preparation: Prior to use, inspect all buffers for precipitation which can occur due to cold storage or prolonged shelf life. If precipitates are observed, gently warm the buffers to 37°C, stirring until the solids have dissolved. Cool the buffers to room temperature before application to prevent thermal degradation of DNA.

Centrifugation Parameters: Follow the kit's specified centrifugation speeds and times rigorously. These parameters are optimized to ensure maximum recovery of DNA while effectively separating it from proteins, lipids, and other cellular debris. Deviations might lead to lower yields or contamination of the eluted DNA.

Maximum Capacity: To prevent column clogging and ensure efficient DNA purification, do not exceed the recommended sample volume and loading capacity of the spin columns. Overloading can lead to incomplete binding of DNA to the column or carryover of impurities.

Component Stability: Proper storage of kit components is critical for maintaining their efficacy. Store enzymes and sensitive reagents at temperatures specified in the kit documentation to preserve their activity and shelf life. Most reagents in this kit are stable at room temperature, but always check the label for specific storage instructions.

Concentration and Yield: The elution volume can be adjusted based on the desired concentration. A smaller volume results in higher concentration but may reduce overall yield. It's important to balance these factors based on the requirements of subsequent applications.

Optimal Recovery: For optimal recovery, ensure that the elution buffer is in direct contact with the entire surface of the silica membrane by allowing it to incubate on the bench for 2 minutes before centrifuging during the elution step.

Technical Support: Gene Vantage offers comprehensive technical support. If you encounter any issues or have questions about the kit's usage, do not hesitate to contact our technical support team. We are here to help you achieve the best possible results with our products.

3. SAFETY PRECAUTIONS

Ensure the safety of all laboratory personnel by adhering to standard laboratory practices when using the IsoBind blood and tissue extraction kit.

When working with chemicals, always wear a suitable lab coat, disposable gloves, and protective goggles. Guanidine salts can form highly reactive compounds when combined with bleach. If liquid containing these buffers is spilt, clean with suitable laboratory detergent and water. If the spilt liquid contains potentially infectious agents, clean the affected area first with laboratory detergent and water, and then with 1% (v/v) sodium hypochlorite.

Many of the reagents included in the kit are chemical in nature and should be handled in a well-ventilated area. Users should be familiar with the safety data sheets (SDS) for each chemical component for information on potential hazards and first aid measures in case of accidental exposure.

Treat all samples as potentially infectious material. Following the universal precautions for handling biological materials will help protect not only the individual conducting the experiment but also the wider laboratory environment.

Dispose of all waste materials according to your institution's safety guidelines and regulations. This includes the proper disposal of used reagents, consumables, and biological waste to mitigate any potential hazards.

CAUTION: DO NOT add bleach or acidic solutions directly to the sample preparation waste.

4. KIT PRINCIPLES

The IsoBind Blood and Tissue Kit is engineered to efficiently isolate high-quality DNA from blood and tissue samples using a robust process grounded in the solid-phase extraction principle. This process is facilitated by silica-based spin columns specifically designed to maximize DNA yield and purity. Below is a detailed explanation of each step involved in the DNA extraction process:

Cell Lysis: The first crucial step involves the breakdown of cell membranes to release DNA into the solution. Effective lysis is key to ensuring that all DNA is accessible for subsequent binding. This kit uses Lysis Buffer, which contains a combination of surfactants and a buffering agent that disrupts cellular and nuclear membranes. The lysis reaction is enhanced by incubating the sample mixture at 56°C which ensures complete lysis.

DNA Binding: Following lysis, the free DNA must be selectively captured or bound while other cellular debris and impurities are excluded. DNA in the lysate binds to a silica membrane within the spin column when in the presence of the Binding Buffer. This environment promotes the adherence of DNA to the silica surface due to the formation of hydrogen bonds between the negatively charged phosphate groups of the nucleic acids and the positively charged surface of silica. This step is critical as it determines the yield of DNA.

Washing: Clean DNA is essential for sensitive applications like PCR. The kit contains two sequential wash buffers (Wash A, and B) each designed to efficiently remove different types of contaminants. Wash Buffer A primarily removes proteins and other large organic molecules, Wash Buffer B is designed to eliminate smaller molecules and salts. Each wash involves adding a specific volume of buffer, followed by centrifugation to pull the liquid through the column while the DNA remains bound to the silica membrane. This ensures that only purified DNA remains on the column.

Elution: The final step is to release the purified DNA from the silica membrane for use in downstream applications. Elution is achieved by applying an Elution Buffer, which disrupts the hydrogen bonds between the DNA and the silica, allowing the DNA to be released into the buffer. The elution buffer is pre-warmed to enhance the efficiency of DNA recovery. The DNA is collected by centrifugation, which forces the eluted DNA into a clean microcentrifuge tube.

Key Features:

Quality of Output: This kit utilizes robust silica-based spin column technology, which selectively binds DNA while efficiently removing contaminants. This results in DNA with high purity, characterized by optimal A260/A280 ratios, indicating minimal protein contamination and readiness for sensitive downstream applications.

Comprehensive Cell Disruption: The Lysis Buffer effectively disrupts cell types found in blood and tissue samples, ensuring complete release of DNA.

Enhanced Recovery: Tailored for samples that are difficult to lyse, the system ensures that even tightly bound genomic material is made available for extraction, which is critical for achieving consistent results across different sample types.

Ease of Use: The protocol is designed to be straightforward with clear step-by-step instructions, reducing the potential for operator error and the need for extensive training.

Streamlined approach: Specifically optimized for blood and tissue samples, including whole blood, buffy coat, body fluids, and various tissue types (both fresh and frozen) the kit's robust lysis and binding conditions are effective in isolating high quality DNA, making it a useful tool in both clinical diagnostic and research settings.

Time efficiency: The entire DNA extraction process can be completed in approximately 45 minutes for 12-24 samples, which is ideal for labs seeking to maintain their turnaround times without compromising on the quality of results.

Compatibility with Downstream Applications: The high-quality DNA extracted is suitable for a variety of molecular biology techniques, including PCR, qPCR, and next-generation sequencing, ensuring broad applicability.

Scalability: The kit is suitable for both low and high volume sample processing, with options for manual (single spin column) and semi-automated (96 well spin plate) workflows.

Note: Please engage with Gene Vantage technical support (see above: Important notes) should you require higher throughput.

5. HARDWARE AND CONSUMABLES (SUPPLIED BY THE USER)

5.1 Hardware

Centrifuge: A high-speed centrifuge capable of achieving at least 13,000 x g is essential for the effective sedimentation of cellular debris and the precise separation of supernatants during the DNA extraction process. The centrifuge must be reliable and capable of maintaining consistent speeds to avoid variations that could affect the purity and yield of the extracted DNA. A temperature control feature to protect sensitive samples from heat degradation during extended spin cycles.

Vortex Mixer: A vortex mixer is required to thoroughly mix samples with lysis and binding buffers, which is crucial for the complete lysis of cells and the homogeneous suspension of DNA within the solution. This ensures maximum contact between the DNA and the silica binding surface, increasing the efficiency of DNA recovery.

Thermomixer/ heating block/ oven:

Required for the incubation of samples at controlled temperatures during the lysis and elution steps. The ability to set precise temperatures is essential, as optimal lysis conditions can vary depending on the sample type and the specific requirements of the DNA extraction protocol.

5.2 Consumables

Microcentrifuge Tubes (1.5 mL): Used for sample preparation and for collecting the eluted DNA.

Pipettes and Aerosol-Barrier Pipette Tips: Precision pipettes and aerosol-barrier tips are crucial for the accurate measurement and transfer of fluids, which is vital for maintaining the correct buffer ratios and avoiding cross-contamination between samples. This is particularly important when working with infectious agents or when performing multiple extractions to ensure reproducible and reliable results. The pipettes should be regularly calibrated to ensure accuracy, and the tips should be certified DNase-free to prevent the degradation of DNA by residual enzymatic activity.

Ethanol (96-100%, molecular grade): Added to wash buffers to help in washing away impurities without stripping the DNA from the column.

Isopropanol (95%, molecular grade): Added to the binding buffer to improve the yield and quality of DNA by ensuring more efficient binding of DNA to the column.

6. QUICK VIEW PROTOCOL

Steps	Procedure	Details
Sample Preparation (if applicable)	Crush or mince tissue samples manually with clean instruments or homogenize using a bead mill.	For blood samples, ensure they are fully mixed and free from clots.
Lysis	Add 400 µL Lysis Buffer to sample > vortex> Incubate at room temperature for 10 mins	Check Lysis Buffer for any precipitate and dissolve if necessary by heating.

Binding	Add an equal volume of Binding Buffer to lysate > Apply mixture to spin column > centrifuge @ 8,000 x g for 1 mins	Volume of lysate to binding buffer should be 1:1 i.e 500 μL: 500 μL
Washing	Add 700 μL of Wash Buffer A >centrifuge at 8,000 x g for 1 mins > Repeat with Wash Buffer B.	Ensure all wash buffer passes through column
Dry Centrifuge	Dry the column	After the final wash, centrifuge at maximum speed for 1 minute to ensure no residual wash buffer remains in the column.
Elution	Place column in collection tube> Add 50 μL of Elution Buffer > incubate @ room temperature for 2 mins> before centrifuging for 1 min at 8,000 x g	
Storage	Eluted DNA is suitable for immediate use or for storage	Store eluted RNA at -20 degrees for short term storage and -80 degrees for long term storage

7. KIT SPECIFICATIONS

Parameter	Specification
Format	Silica spin columns
Sample Material	Blood, tissue
Typical Yield	Up to 30 µg per isolation
Purity Ratios	A260/A280 > 1.8
Elution Volume	50 µL
Preparation Time	Approx. 45 minutes
Binding Capacity	Up to 100 µg DNA

8. WORKFLOW TIPS

To maximize the effectiveness and reliability of the IsoBind blood and tissue Kit, it is crucial to consider additional aspects of the extraction process that impact both the quality of the DNA obtained and the user's experience. These additional suggestions provide guidance on sample quality and preparation, elution efficiency, and quality control measures:

COLLECTION AND STORAGE OF STARTING MATERIAL

Immediate Processing: Ideally, samples should be processed immediately after collection to minimize DNA degradation. If immediate processing is not possible, samples must be handled and stored carefully to preserve their integrity.

Blood Samples: Blood samples should ideally be processed within 2 hours of collection to prevent the breakdown of cellular components and to maintain the integrity of the DNA. If immediate processing is not possible, keep the blood at room temperature and proceed to storage guidelines as soon as feasible.

Tissue Samples: Process tissue samples as soon as possible after collection to minimize enzymatic degradation of DNA by endogenous nucleases. If immediate processing is not viable, proceed to freezing as soon as possible, place tissue samples directly in liquid nitrogen to rapidly freeze them, preserving cellular structure and DNA integrity. If liquid nitrogen is not available then freeze according to storage considerations.

Storage Considerations: For short-term storage refrigerate blood samples at 2°C to 8°C immediately after collection. Samples stored under these conditions are stable for up to 7 days for long-term storage; blood samples should be frozen. Freeze samples at -20°C for up to 6 months. For periods longer than 6 months, consider storing at -80°C to better preserve DNA quality.

Handling: Avoid multiple freeze-thaw cycles as they can significantly degrade DNA quality. Thaw frozen samples on ice when ready for processing.

SAMPLE SIZE CONSIDERATIONS

Optimal Sample Volume Blood Samples: For most applications, a blood sample volume of 200 to 500 μ L is optimal. This volume is sufficient to achieve a good yield of DNA while ensuring efficient processing.

Column Capacity Blood Samples: Ensure that the volume of the blood sample does not exceed the capacity of the silica spin columns provided in the kit. Overloading the column can result in lower DNA purity and yield because the column may not effectively bind all the DNA from an excessively large volume.

Optimal Sample Volume for Tissue Samples: For tissue samples, up to 25 mg per extraction is recommended. This amount typically provides a sufficient quantity of DNA for downstream applications without overloading the extraction system. Optimal sample size and weight are not only about adhering to technical limits but also about maximizing the efficiency of DNA recovery. Too small a sample might not yield enough DNA for your needs, whereas too large a sample can strain the capacity of the kit components, leading to inefficient extraction and potential loss of material.

Sample Homogeneity for Tissue Samples: To ensure consistent DNA yield and quality, the tissue sample should be homogeneous. This means the tissue should be evenly divided and representative of the whole sample to avoid variability in the results.

Handling Larger Sample Volumes for Tissue Samples: For samples exceeding the standard 1 ml size, it may be necessary to process the material in multiple batches or adjust the protocol to accommodate larger volumes, which includes scaling up the volumes of all reagents and possibly using multiple spin columns.

Concentration and Yield for Tissue Samples: Adjust the volume of Elution Buffer based on the required concentration of DNA; smaller volumes yield more concentrated DNA, which might be crucial for sensitive downstream applications like PCR.

9. PREPARING BUFFERS AND EQUIPMENT

Before Starting:

Centrifuges: Before beginning any procedures, ensure that the centrifuge is functioning correctly. Perform a test run to check for any unusual noises or vibrations that could indicate a maintenance issue. Ensure that the rotor is securely fastened and that the lid closes properly.

Calibration: Regular calibration of the centrifuge is crucial for achieving the precise speeds necessary for optimal DNA isolation. Inaccuracies in speed can lead to inefficient separation of phases, potentially contaminating the DNA sample or resulting in lower yields.

Cleaning: Clean the centrifuge and rotor regularly to prevent the buildup of dust and biological material, which could interfere with operations or contaminate samples. Use appropriate disinfectants to wipe down the interior and rotor, especially after handling potentially infectious samples.

Pipettes: Verify the accuracy of all pipettes before use. This can be done by pipetting distilled water onto a precision scale to check if the dispensed volumes are within the manufacturer's specified tolerance.

Calibration: Calibrate pipettes regularly according to the manufacturer's guidelines to ensure they dispense volumes accurately, which is critical for the precise preparation of buffers and reagents.

Maintenance: Clean pipettes frequently to prevent cross-contamination between samples. Check the pipette tips for any residual sample before each use, and replace pipette tips between samples to maintain sample integrity.

Vortex Mixer: Ensure that the vortex mixer is operating correctly. Test the mixer by running it at different speeds to ensure it can provide the vigorous agitation needed for thorough mixing of lysis buffers with samples.

Stability: Check the stability of the vortex mixer on the bench to prevent any movement during operation, which could affect the homogeneity of sample mixing.

Balances: Regularly check and calibrate balances used to weigh samples or reagents to ensure precision. Incorrect measurements can alter the concentration of reagents, affecting the efficiency of the DNA extraction.

Cleanliness: Keep the balance area clean and free from vibrations and drafts, which could affect the accuracy of measurements.

Preparation: Prepare all consumables in advance by arranging them in an orderly manner on the workstation. This organization helps prevent confusion and potential contamination during the extraction process. Ensure that all reagents are within their expiration dates and have been stored under the correct conditions. Any reagent that appears cloudy or precipitated should be warmed gently, if permissible, and mixed thoroughly to redissolve any solids.

Workspace Preparation: Disinfect the workspace thoroughly before starting the extraction to create an RNase-free environment. Use DNase decontamination solutions and maintain clean bench practices throughout the procedure.

10. COMPLETE PROTOCOL

1. Sample Preparation:

- 1.1 For blood samples: Transfer 200 µl of whole blood into a clean centrifuge tube. Ensure the blood is fresh or properly anticoagulated to prevent clotting.
- 1.2 For tissue samples: Finely chop up to 25 mg of tissue using a sterile scalpel or scissors to increase the surface area exposed to lysis reagents, promoting complete lysis.
- 1.3 Transfer the chopped tissue into a microcentrifuge tube.

2. Lysis:

- 2.1 Add 20 µl of Proteinase K and 200 µl of Lysis Buffer to the sample in the tube. Proteinase K helps degrade proteins and aids in nucleic acid release.
- 2.2 Vortex the tube vigorously to ensure thorough mixing and exposure of the entire sample to the lysis reagents.
- 2.3 Incubate the mixture at 56°C. For tissue samples, incubation may last 1-3 hours depending on the tissue type and its density; blood typically requires less time.

3. Binding DNA:

- 3.1 Add 200 µl of ethanol to the lysate and mix thoroughly by vortexing. Ethanol precipitates DNA, making it suitable for binding to the silica membrane in the spin-column.
- 3.2 Transfer the entire mixture to a spin-column placed in a collection tube. Centrifuge at 6000 x g for 1 minute.

4. Washing:

- 4.1 Add 500 µl of Wash Buffer A to the column.
- 4.2 Centrifuge at 6000 x g for 1 minute and discard the flow-through. This buffer removes the bulk of the residual proteins and other contaminants.
- 4.3 Add 500 µl of Wash Buffer B
- 4.4 Repeat the centrifugation for 3 minutes.

5. Dry Centrifuge

- 5.1 After the final wash, centrifuge at maximum speed for 1 minute to ensure no residual wash buffer remains in the column.

6. Elution of DNA:

- 6.1 Transfer the spin-column to a clean centrifuge tube to avoid cross-contamination.
- 6.2 Add 200 µl of Elution Buffer directly onto the membrane of the spin column. Let it incubate for 1 minute at room temperature to allow the DNA to dissociate from the column matrix.
- 6.3 Centrifuge at 6000 x g for 1 minute to collect the eluted DNA in the tube. The gentle elution conditions help maximize the recovery of high-quality DNA.

7. Storage of DNA:

- 7.1 Store the eluted DNA at -20°C.

11. TROUBLESHOOTING GUIDE

Problem Description	Possible Causes	Suggestions
Low DNA Yield	Incomplete lysis of sample	Ensure thorough homogenization of the sample. Increase the amount of Lysis Buffer if necessary.
	Insufficient incubation time	Extend the incubation time during the lysis step.
	DNA degradation	Use fresh samples or store samples properly. Add RNase inhibitors if needed.
Poor RNA purity (low A260/A280 ratio)	Contamination with proteins or phenolic compounds	Increase the number of wash steps. Ensure complete removal of Wash Buffers.
	Incomplete removal of wash buffers	Perform additional dry centrifugation steps to remove residual wash buffers.
DNA Degradation	Improper sample storage	Store samples at 4-8°C for short term storage and at -20 to -80°C for long term storage or use immediately after collection.
	Prolonged exposure to room temperature	Keep samples and lysates on ice during the extraction process.

Inconsistent DNA yield	Variation in sample size or type	Standardize the amount and type of starting material.
	Inconsistent execution of the protocol	Follow the protocol steps precisely and consistently.
Clogged spin column	Excessive sample material	Reduce the amount of starting material or increase the volume of Lysis Buffer.
	Insufficient centrifugation	Increase the centrifugation speed or duration.
Contamination in DNA samples	Cross-contamination between samples	Use sterile equipment and consumables. Practice good laboratory hygiene.
	Contamination of reagents or equipment	Use fresh reagents and clean equipment before use.
Equipment Malfunction	Centrifuge not reaching required speed	Check the centrifuge settings and performance. Calibrate or repair the centrifuge if necessary.
	Pipettes delivering inaccurate volumes	Calibrate pipettes regularly. Use pipettes with the correct volume range for the protocol.
Buffer Precipitation	Cold storage of buffers that should be at room temperature	Ensure all buffers are stored according to the specifications provided in the manual. Label storage containers clearly with the appropriate storage temperatures and routinely check storage conditions.
	Incorrect preparation of buffers	Adhere strictly to the buffer preparation instructions provided in the manual. Measure all components accurately and mix thoroughly

Difficulty in Eluting DNA	Spin column membrane dried out	Pre wet the column before adding the elution buffer with a few micro litres of RNase free water. Ensure the spin column does not sit out for too long after the final wash step.
----------------------------------	--------------------------------	--

12. PRODUCT USE RESTRICTION / WARRANTY

GENE VANTAGE kit components are intended, developed, designed, and sold for research purposes only. All kit components are for general laboratory use only and should only be used by qualified personnel wearing the appropriate protective clothing. GENE VANTAGE does not assume any responsibility for damages due to improper application of our products in other fields of application. Any user, whether by direct or resale of the product, is liable for any and all damages resulting from any application outside of research.

There is no warranty for and GENE VANTAGE is not liable for damages or defects arising in shipping and handling, or out of accident or improper or abnormal use of this product; defects in products or components not manufactured by GENE VANTAGE, or damages resulting from such non-GENE VANTAGE components or products. GENE VANTAGE makes no other warranty of any kind whatsoever, and specifically disclaims and excludes all other warranties of any kind or nature whatsoever, directly or indirectly, express or implied, including without limitation as to the suitability, reproductivity, durability, fitness for a particular purpose or use, merchantability, condition, or any other matter with respect to GENE VANTAGE products.

GENE VANTAGE shall only be responsible for the product specifications and the performance range of GENE VANTAGE products according to the specifications of in-house quality control, product documentation and marketing material. This GENE VANTAGE product is shipped with documentation stating specifications and other technical information. GENE VANTAGE's sole obligation and the customer's sole remedy is limited to replacement of products free of charge in the event products fail to perform as warranted.

In no event shall GENE VANTAGE be liable for claims for any other damages, whether direct, indirect, incidental, compensatory, foreseeable, consequential, or special (including but not limited to loss of use, revenue or profit), whether based upon warranty, contract, tort (including negligence) or strict liability arising in connection with the sale or the failure of GENE VANTAGE products to perform in accordance with the stated specifications. This warranty is exclusive and GENE VANTAGE makes no other warranty expressed or implied.

Applications mentioned in GENE VANTAGE literature are provided for informational purposes only. GENE VANTAGE does not warrant that all applications have been tested in GENE VANTAGE laboratories using GENE VANTAGE products. GENE VANTAGE does not warrant the correctness of any of those applications.